



www.grassinstruments.com

New Installation & Upgrade Procedure: PolyVIEW® v2.5 Review Module

Note: It is critical that you follow the order of the installation procedure to properly install PolyVIEW v2.5. PolyVIEW v2.5 is designed to run on Windows 98/2000/NT4.

Contents of **PVReviewInstallation(T005C98).wri**

Step 1. PolyVIEW v2.5 Review Module Installation

Step 2. PolyVIEW v2.5 Review Module Patch1 Installation

Step 1. PolyVIEW v2.5 Review Module Installation

1. Insert the PolyVIEW Review-CD into the CD-ROM drive.
2. Start > Run... The Run popup window is displayed.
3. On the Open: line, type the following: [CD-ROM drive letter]:\Review_Module\Setup.exe
Click OK. The PolyVIEW Review Module Installation program will run.
4. Follow the on screen instructions, accepting all the default settings.
5. At the completion of the installation, LabVIEW Run-Time Engine will install, which can take up to several minutes. When the installation is complete a popup will state that the Installation was Successful. Click OK.

Step 2. PolyVIEW v2.5 Review Module Patch1 Installation

This installation will update the Reviewer.llb in your C:\Pvreview directory. To check which version you have, open the 'About PolyVIEW' window. After performing this installation, the date of the Reviewer.llb should be 10/5/2001.

1. Insert the CD-ROM.
2. Start > Run...
3. On the Open: line type: [CD-ROM Drive Letter]:\Extras\Patch1.exe
Click OK. The 'WinZip Self-Extractor - Patch1.exe' window is displayed. Click Unzip.
4. When it is finished the 'WinZip Self-Extractor' popup will state: '2 file(s) unzipped successfully.' Click OK.
5. In the 'WinZip Self-Extractor - Patch1.exe' window, click Close.
6. Reboot your computer.

To Run PolyVIEW, Start > Programs > Grass Instruments > PolyVIEW Review